# Doro 9350

## Blow alarm

### ENGLISH

Connect this device to the radio transceiver to communicate wirelessly between device and carephone. In order for the device to send alarms through the carephone, you need to pair them.

#### How to pair the radio transceiver with the carephone

1. Connect the **9350** to the **blow**.
2. **On the carephone**; press and hold the **green button** for four seconds until a beep sounds to start pairing mode. Release the button.
3. **On the blow**; blow into the the blow switch to initiate pairing. A confirmation beep is heard on the carephone.
4. **On the carephone**; press the **alarm button** to assign the new device. A new confirmation beep is heard.

For more information on pairing the carephone with the radio transceiver, refer to the manual of the carephone.

#### How to test the alarm after pairing

**On the blow**; blow into the blow switch to generate an alarm. Check that the alarm is received at the Alarm Receiving Center.

#### This is how it works

1. When an alarm is triggered on the device, the radio transceiver sends the alarm to the carephone.
2. The carephone passes the alarm to the alarm receiving centre or another receiver.

#### Battery warning

The device automatically alarms at low battery. Replace the battery as soon as possible after low battery level has been indicated.

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