# Doro 9350

## Epilepsy sensor

### ENGLISH

In this device we have integrated a radio transceiver to communicate wirelessly between device and carephone. In order for the device to send alarms through the carephone, you need to pair them.

#### How to pair the radio transceiver with the carephone

1. Attach the bed movement sensor, connect power and turn on device.
2. **On the epilepsy sensor**; wait for the green LED to stop flashing.
3. **On the epilepsy sensor**; set the **Movement Sensitivity to medium** and set the **Time delay to 0**.
4. **On the carephone**; press and hold the **green button** for four seconds until a beep sounds to start pairing mode. Release the button.
5. **On the epilepsy sensor**; simulate movements by tapping the sensor until a confirmation tone is heard from carephone.

6. **On the carephone**, press the **alarm button** to assign the new device. A new confirmation beep is heard.

For more information on pairing the carephone with the radio transceiver, refer to the manual of the carephone.

#### How to test the alarm after pairing

**On the epilepsy sensor**; Simulate movements by tapping the sensor. Check that the alarm is received at the Alarm Receiving Centre.

#### This is how it works

1. When an alarm is triggered on the device, the radio transceiver sends the alarm to the carephone.
2. The carephone passes the alarm to the alarm receiving centre or another receiver.

#### Battery warning

The device automatically alarms at low battery. Replace the battery as soon as possible after low battery level has been indicated.

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©2017 Doro AB. All rights reserved.